

DETERGENT DISINFECTANT FOR FLOORS AND SURFACES

SURFANIOS Premium

DETERGENT DISINFECTANT FOR FLOORS AND SURFACES

Cleaning and disinfection of non-invasive medical devices prior to use and according to their instructions for use.
Biocide: for general surfaces.

- Twin action: combination of detergent and disinfectant action in a simultaneous operation
- Broad efficacy spectrum in short contact times
- Readily biodegradable chelating agent

**ECOLAB®**www.ecolab.com

LABORATOIRES
ANIOS
An Ecolab Company

www.anios.com

SURFANIOS Premium

Detergent disinfectant for floors and surfaces



Surfanios Premium
5 L Can



Surfanios Premium
1 L Bottle



Surfanios Premium
20 ml Sachet

PRODUCT CHARACTERISTICS

- Green coloured solution
- pH of pure product: 10.9
- pH of diluted product (0.25%): approx 8.5
- Density at 20°C: 1.009 g/ml
- Refraction index: 1.3587

MICROBIOLOGICAL PROPERTIES

EFFICACY	CONCENTRATION	CONTACT TIME
Bactericidal - According to: EN 13727* (5 min 0.25%), EN 16615* (15 min, 0.25%)	0.25 %	15 min
Yeastcidal - According to: EN 13624* (15 min, 0.25%), EN 16615* (15 min, 0.25%)	0.25 %	15 min
Tuberculocidal - According to EN 14348*	0.50 %	30 min
Effective against enveloped viruses according to EN 14476*	0.25 %	5 min
Limited spectrum virucidal according to EN 14476*	2.0 %	30 min
EN 14476* Adenovirus	1.0 %	30 min
EN 14476* Norovirus (MNV)	1.0 % / 2.0 %	60 min / 30 min

*dirty conditions

ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
SURFANIOS Premium	4 x 5 L	1917.036
SURFANIOS Premium (with integrated 20 ml dosing chamber)	12 x 1 L	1917.092
SURFANIOS Premium Sachets	500 x 20 ml	1917.129



LABORATOIRES ANIOS
1, Rue de l'Espoir
59260 Lezennes
France

Distributed by:
LABORATOIRES ANIOS
1, Rue de l'Espoir
59260 Lezennes
France



www.ecolab.com



www.anios.com

APPLICATION AREA

Detergent disinfectant for floors and surfaces.
Cleaning and disinfection of non-invasive medical devices prior to use and according to their instructions for use.
Biocide: for general surfaces.

INSTRUCTIONS FOR USE

Concentrated solution.

Dilute the concentrate with water to the desired concentration.

To dilute e.g. to a concentration of 0.25% (2.5 ml/l), add 20 ml for 8 litres of cold or hot water (max. + 60 °C).

Apply in sufficient quantity (+/- 30 ml/m²).

Respect the recommended contact time for the required antimicrobial activity.

For medical devices: rinse thoroughly with tap water to remove any trace of the product.

For biocide applications: Rinsing is unnecessary except for the areas to be treated that come in contact with the skin.

For the frequency of use and cleaning of the application equipment, refer to the hygiene plan in place.

Scientific dossier (efficacy, compatibility, etc.) available on request.

INGREDIENTS

N-(3-aminopropyl)-N-dodecylpropane-1,3-diamine (CAS N° 2372-82-9: 51 mg/g), Didecyltrimethylammonium chloride (CAS N° 7173-51-5: 25 mg/g), non-ionic surfactants, sequestering agents, colorant, perfume and excipients.

CERTIFICATES

Medical device and Biocide.
Full antimicrobial activity for SURFANIOS Premium is included in the scientific dossier, available on request.

STORAGE

Storage: between +5°C and +25°C.

MEDICAL DEVICE AND BIOCIDES USE SAFELY

Please read label and product information.



FOR PROFESSIONAL USE ONLY.